

**Taurolidine Lock  
Citrate-free!**

**NutriLock<sup>TM</sup>**  
CE 0123

**ANTIMICROBIAL CATHETER LOCK SOLUTION  
FOR PARENTERAL NUTRITION**



# NutriLock™

CE 0123

## INDICATION:

NutriLock™ is indicated for those patients who use a port or a silicone or polyurethane catheter-based device as vascular access for parenteral nutrition and medication. NutriLock is intended to be used as a catheter lock solution in order to maintain patency of the vascular access device. Additionally, by creating a hostile environment to bacterial and fungal growth, the risk of catheter-related bloodstream infections (CRBSI) is reduced.

## INGREDIENTS / SPECIFICATION:

NutriLock™ contains the active ingredient taurolidine (2%) which provides a broad antimicrobial activity against gram-positive and gram-negative bacteria (incl. resistant strains like MRSA) and fungi (e.g. Candida). Note: In contrast to other locking solution of our product range NutriLock™ does not contain citrate.

NutriLock™ is available as a box of 10 x 3 mL ampoules. Each ampoule is for single use.

## INSTALLATION:

After infusion of parenteral nutrition the system is thoroughly flushed with saline (min. 10 mL). Then, NutriLock™ is instilled into the access system by exactly respecting the filling volume. NutriLock™ remains in the system until the next nutrition session. NutriLock™ must be aspirated from the system before starting a nutrition session. If aspiration of NutriLock™ is not possible, slow flushing of NutriLock™ (not more than 1 mL per 3 seconds) prior to the next treatment is clinically tolerable. Flush the device with 10 mL of saline.

## CONTRAINDICATIONS:

NutriLock™ is contraindicated for patients with known allergy to taurolidine and for patients currently taking drugs that have demonstrated to cause undesirable interactions with taurolidine.

## STORAGE:

NutriLock™ must be stored at 15 to 25°C. NutriLock™ must be neither transported nor stored under freezing conditions.

**NOTE:** This information does not replace the Instructions for use.



Manufacturer:



TauroPharm GmbH  
August-Bebel-Str. 51  
D-97297 Waldbüttelbrunn  
Tel.: +49 (0) 931 304 299-0

ISO 13485

[WWW.TAUROLOCK.COM](http://WWW.TAUROLOCK.COM)

85107/49/18